**STOPS BLEEDING
INSTANTLY!**

WOUNDSEAL®

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**Topical Powder
For Minor External Bleeding**

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POLVO + PRESIÓN = COSTRA INSTANTÁNEA

Package contains four (4) single use applications in tubes.

INSTRUCTIONS FOR USE:

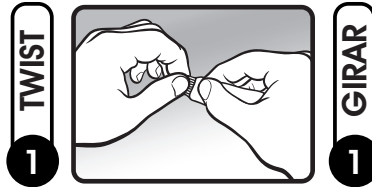
If possible, clean wound and apply powder after bleeding resumes; blood must be present for powder to work.

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
**TWIST CAP TO REMOVE.
PUNCTURE FOIL COMPLETELY
WITH TIP OF CAP TO OPEN.**

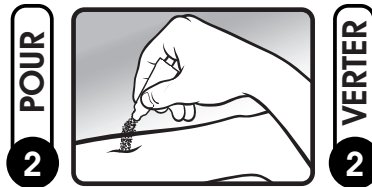**INVERT TUBE TO POUR POWDER
ONTO WOUND. COVER WOUND
COMPLETELY WITH POWDER.**

**PRESS FOR 30 SECONDS,
UNTIL SCAB FORMS AND
BLEEDING STOPS.**

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Repeat application as needed. The protective scab may be left in place until it falls off naturally. Avoid wetting scab for one (1) hour.

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WOUND COVERAGE AREA: One WoundSeal tube is sufficient to cover a wound area up to 3.9 cm² (0.6 in²) to a depth of 2 mm.

INGREDIENTS: Hydrophilic polymer and potassium ferrate

WARNINGS: For external use only. Not intended to replace stitches. Do not apply powder to visibly infected wounds. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Keep out of reach of children.

CAUTIONS: Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged, opened, or if powder has clumped and does not flow freely. Discard contents after use since powder is guaranteed to work only if it has not been left exposed to air or humidity. Occasionally, a temporary warming or stinging sensation may be felt upon application.

PRODUCT REMOVAL: If WoundSeal Powder must be removed, as determined by a healthcare professional, brush away loose powder, then soak wound in sterile water until scab starts to loosen from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

SYMBOLS GLOSSARY: www.biolife.com/symbols

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