**STOPS BLEEDING  
INSTANTLY!**

# WOUNDSEAL<sup>®</sup>

## +APPLICATOR

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**Topical Powder  
For Minor External Bleeding**

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**POWDER + PRESSURE = INSTANT SCAB**

**POLVO + PRESIÓN = COSTRA INSTANTÁNEA**

**TAMPER EVIDENT: Do not use if blister seal is damaged.**

Package contains four (4), single use applications in pouches. Enclosed applicator swab can be used to apply WoundSeal Powder to hard-to-reach wounds, or to apply powder to another person. Each pouch contains one (1) single use application which includes one (1) applicator and one (1) blister pack of WoundSeal Powder.

### INSTRUCTIONS FOR USE:

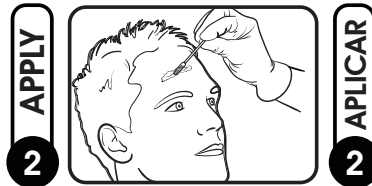
If possible, clean wound and apply powder after bleeding resumes; blood must be present for powder to work.

**PEEL BLISTER AND LOAD  
APPLICATOR WITH POWDER.**

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**COVER WOUND COMPLETELY  
WITH POWDER.**

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**PRESS FOR 30 SECONDS, USING  
APPLICATOR OR FINGERS, UNTIL SCAB  
FORMS AND BLEEDING STOPS.**

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Repeat application as needed. The protective scab may be left in place until it falls off naturally. Avoid wetting scab for one (1) hour.

**WOUND COVERAGE AREA:** One (1) WoundSeal application is sufficient to cover a wound area of up to 4.6 cm<sup>2</sup> (0.7 in<sup>2</sup>) to a depth of 2 mm.

**INGREDIENTS:** Hydrophilic polymer and potassium ferrate

**WARNINGS:** For external use only. Not intended to replace stitches. Do not apply powder to visibly infected wounds. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Keep out of reach of children.

**CAUTIONS:** Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged, opened, or if powder has clumped and does not flow freely. Discard contents after use since powder is guaranteed to work only if it has not been left exposed to air or humidity. Occasionally, a temporary warming or stinging sensation may be felt upon application.

**PRODUCT REMOVAL:** If WoundSeal Powder must be removed, as determined by a healthcare professional, brush away loose powder, then soak wound in sterile water until scab starts to loosen from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

**SYMBOLS GLOSSARY:** [www.biolife.com/symbols](http://www.biolife.com/symbols)

For more information or to share feedback, visit [www.woundseal.com](http://www.woundseal.com) or call +1 (941) 360-1300.

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